


AClip® Femoral



AClip® F Cage

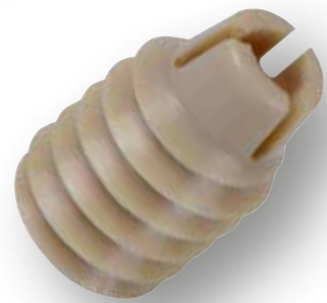

AClip® F Button



## AClip<sup>®</sup> F Cage

- Cancellous cage with retentive strip
- Material: PEEK
- Lg 12 mm Diameter 7.8 mm
- Inserted on a 2.4 mm eylet pin



## AClip<sup>®</sup> F Button

- Titanium plate
- Polyethylene terephthalate loop
- Loop Lg 15 mm
- Dedicated: Cage designed for the Button




## AClip<sup>®</sup> Femoral locking

- Fixation point
  - Controlled: 27-32 or 37 mm
  - Dedicated: Cage designed for the Button
- Instrument locking
- 3 operative steps


### 1 - Drilling: Stepped reamer

- One action = two central tunnels
- Automatic tunnel cage created

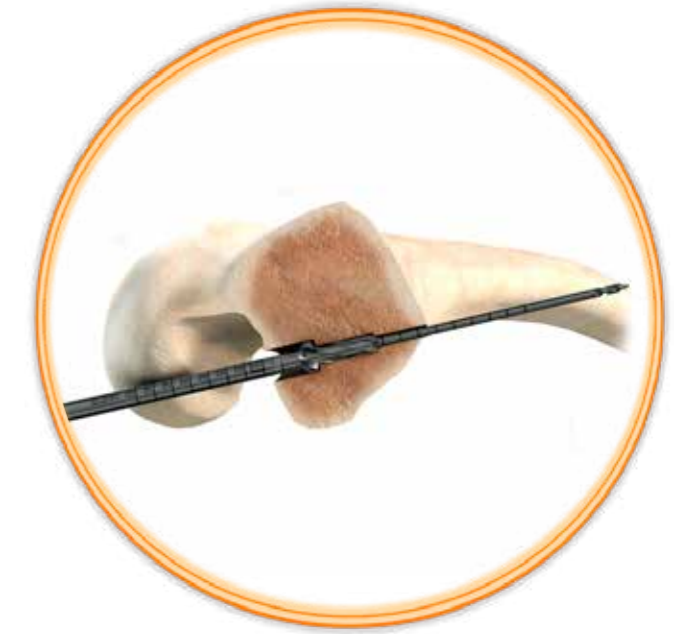


### 2 - Screwing: Cage introducer

- Cage establishment on the eylet pin
- Drilling graduation preservation

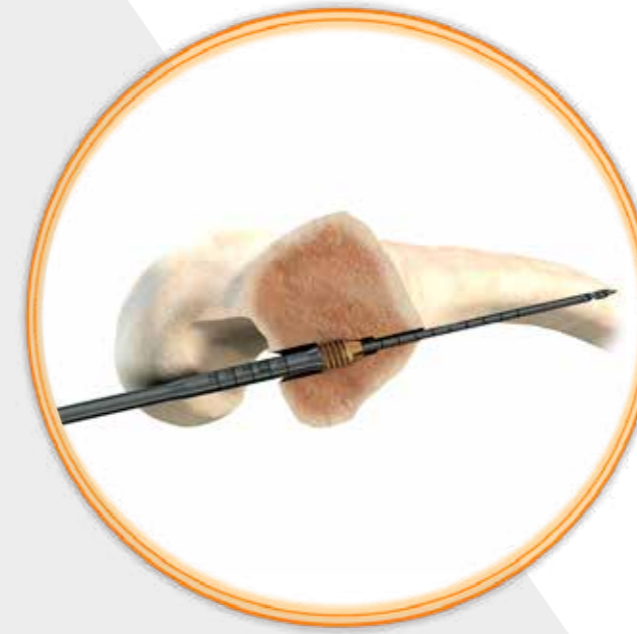

### 3 - Final locking: Tie rod and tensile support

- Progressive implementation
- Locking signal

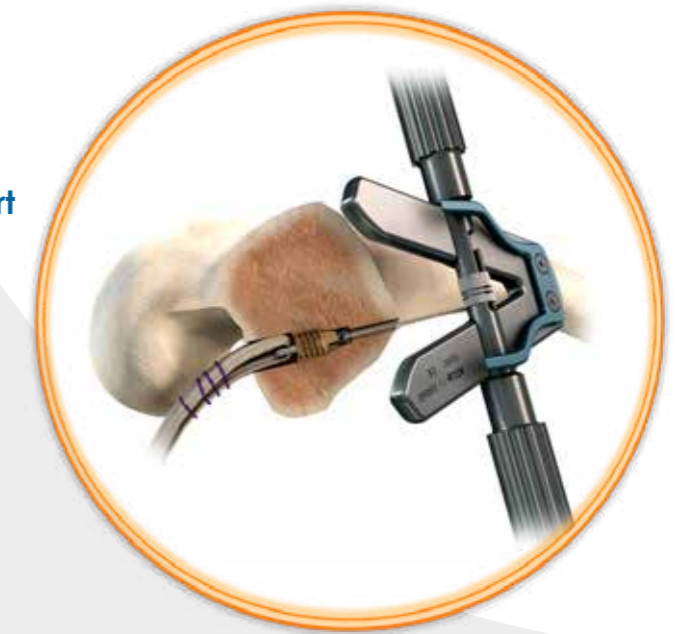

## Specific instrumentation AClip® Femoral



Description	Product nbr	Qty
Tensile support	2-0406600	1
Tie rod	2-0408000	1
Shank introducer AClip® F Cage Trinkle or AClip® F Cage introducer	2-0407200 2-0406000	1 1
Stepped reamer Ø5xØ7 mm	2-0406530	1
Stepped reamer Ø5xØ7,5 mm	2-0406535	1
Stepped reamer Ø5xØ8 mm	2-0406540	1
Stepped reamer Ø5xØ8,5 mm	2-0406545	1
Stepped reamer Ø5xØ9 mm	2-0406550	1
Stepped reamer Ø5xØ9,5 mm	2-0406555	1
Stepped reamer Ø5xØ10 mm	2-0406560	1
Stepped reamer Ø5xØ10,5 mm	2-0406565	1
Stepped reamer Ø5xØ11 mm	2-0406570	1
AClip® eylet pin	2-0404701	1

## AClip® Femoral implant

Description	Product nbr	Qty
AClip® Femoral	1-0402300	1

### Customer Service - France

Porte du Grand Lyon,  
01700 Neyron, France  
Tel. : +33 (0)4 37 85 19 19  
Fax : +33 (0)4 37 85 19 18  
E-mail : [amplitude@amplitude-ortho.com](mailto:amplitude@amplitude-ortho.com)

### Customer Service-Export :

11, cours Jacques Offenbach. Zone Mozart 2,  
26000 Valence, France  
Tel. : +33 (0)4 75 41 87 41  
Fax : +33 (0)4 75 41 87 42  
E-mail : [amplitude@amplitude-ortho.com](mailto:amplitude@amplitude-ortho.com)

Internet : [www.amplitude-ortho.com](http://www.amplitude-ortho.com)